REF 964-0611-9, 974-0581-0, BT-565-104

CA CLEAN™ II

Identification of the IVD reagent
CA CLEAN™ II ;
detergent for fully automated blood coagulation analyzer

Intended use
The product is applied to cleanse the pipets used for fully automated blood coagulation analyzer.

Principles of the examination method
CA CLEAN II is a detergent product used for cleansing the pipets used for our fully automated blood coagulation analyzer.

Components	
Hydrochloric acid	0.16%
Non-ionic surfactant	0.50%

- Warnings and precautions**
- For in vitro diagnostic use only. Never use this on the human body.
 - The product is acidic. Be especially careful when handling it.
 - If eye or mouth contact occurs by mistake, take emergency measures such as washing the area with plenty of water; see a doctor if needed.
 - If you get the detergent on your hands, quickly wash the area with plenty of water.
 - Neutralize the product using plenty of water or other materials when disposing of it.
 - Never freeze the product.
 - Keep the unsealed product at 5 to 35°C and use it within 2 months after unsealing it.
 - Ensure that dust, refuse, or fungi do not get mixed into the product after unsealing it.
 - Do not use an outdated product.

Examination procedure
For actual use of the product, refer to the user instruction manual for the fully automated blood coagulation analyzer.

Storage and shelf life of unopened product
Storage and shelf life after first opening
Store the product from 5 to 35°C (Do not freeze). The expiry date is indicated on both the box and the bottle label. The product is stable for 2 months after unsealing.

Limitations of the examination procedure
The use of this reagent, respectively control or calibrator, is validated on specific analyzers to optimize product performance and meet product specifications. Please refer to Instructions For Use of your analyzer whether the use of this reagent, respectively control or calibrator, is authorised by Sysmex. Sysmex cannot take the responsibility for patient results received from the use of Sysmex reagents, controls or calibrators on unauthorised analyzers. It is the responsibility of the user to validate modifications to these instructions or use of the reagent, control or calibrator on analyzers other than those specified by Sysmex.

Manufacturer	 Sysmex Corporation 1-5-1 Wakinohama-Kaigandori, Chuo-ku, Kobe 651-0073, Japan
Authorized representatives	Americas: Sysmex America, Inc. 577 Aptakisic Road, Lincolnshire, IL 60069, U.S.A. Sysmex Asia Pacific Pte Ltd. Asia-Pacific: 9 Tampines Grande #06-18 Singapore 528735

Product information	
CA CLEAN II (GSZ-500A)	500 mL
CA CLEAN II (GSZ-510A)	5 L
CA CLEAN II (GSZ-520A)	45 mL
Date of issue or revision	
11/2021	

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