CN-COAGWASHER™
Identification of the IVD reagent
CN-COAGWASHER™

Intended purpose
CN-COAGWASHER is a cleaning solution for Sysmex fully automated blood coagulation analyzer, and is used by healthcare professionals and properly trained personnel. For appropriate use of the product, refer to instructions for use of applicable Sysmex fully automated blood coagulation analyzers.

Principles of the examination method
CN-COAGWASHER is a cleaning agent used to clean the pipettes in the Sysmex fully automated blood coagulation analyzers.

Components
Sodium hypochlorite 1.0 % (active chlorine concentration)

Warnings and precautions

- Follow the warnings and precautions, as described on the product container, package box, package insert or device's instructions for use, to handle the product correctly.
 - Avoid direct contact with skin, eyes and mucous membranes. Do not ingest.
 - In case of skin contact, rinse immediately with plenty of water. In case of contact with eyes or mucous membranes, rinse immediately with plenty of water, and seek medical attention. In case of ingestion, seek medical attention immediately.
 - Please see the safety data sheet for the country-specific classification.
- Danger
H290 May be corrosive to metals.
H315 Causes skin irritation.
H318 Causes serious eye damage.
H412 Harmful to aquatic life with long lasting effects.
P280 Wear protective gloves/protective clothing/eye protection/face protection.
P305+P351+P338 IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.
P310 Immediately call a POISON CENTER/doctor.

Additional required equipment

Automated Blood Coagulation Analyzer CN-6000
Automated Blood Coagulation Analyzer CN-3000
Automated Blood Coagulation Analyzer CN-6500
Automated Blood Coagulation Analyzer CN-3500
Consult the instructions for use for the specific devices to be used with.

Examination procedure

Refer to the device's instructions for use.

Storage and shelf life of unopened product
Store the product at 2-35 °C in a dark place (do not freeze). Keep away from humidity, direct sunlight and heat. When the product is properly stored in its sealed container, it is stable until the expiration date printed on the label.

Storage and shelf life after first opening
Once opened the product is stable for 3 months at 2-35 °C. After opening the product container, store in the box and avoid direct contact with sunlight.

Limitations of the examination procedure
The use of this product is validated on specific devices to optimize product performance and meet product specifications. Please refer to the instructions for use of your device to confirm that the use of this product is authorized by Sysmex. Sysmex cannot take the responsibility for patient results obtained from the use of Sysmex products on unauthorized devices. It is the responsibility of the user to validate modifications to these instructions or use of the product on devices other than those specified by Sysmex.

Disposal procedures
Disposal procedures for emptied containers, remaining liquids and waste effluents from the device should meet the requirements of applicable local regulations. Incineration of waste effluents from the device is generally recommended. Waste effluents contaminated with certain reagents must be incinerated. Please refer to the disposal instructions related to the reagents used and dispose the waste effluents in the same way.

Manufacturer
Sysmex Corporation
1-5-1 Wakinohama-Kaigandori,
Chuo-ku, Kobe 651-0073, Japan

Authorized representatives
Asia-Pacific: Sysmex Asia Pacific Pte Ltd.
9 Tampines Grande #06-18, Singapore 528735

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