Fluorocell™ PLT

EN

Identification of the IVD reagent

Fluorocell™ PLT

Intended use

Fluorocell PLT is to be used to label the platelet in diluted blood sample for the assay of platelet count in blood with Sysmex automated hematology analyzers.

Principles of the examination method

A sample volume of a whole blood specimen is introduced into the analyzer where a portion of it is automatically diluted into a 1:200 dilution with CELLPACK DFL. Fluorocell PLT is then added and the entire dilution is maintained at a constant temperature for a defined time period in order to label the platelet present in the sample. The labeled sample is then introduced into the sheath flow detector where forward scattered light and side fluorescence are measured allowing the platelet count (PLT#) to be computed. Consult your specific Sysmex analyzer's Instruction for Use for additional information with respect to procedures and principles for platelet analysis.

Components

Oxazine	0.003%
Ethylene glycol	99.9%

Warnings and precautions (For in vitro diagnostic use only)

- Wear gloves and a lab coat for protection. Avoid contact with skin and eyes.
- In case of skin contact, rinse immediately with plenty of soap and water.
- In case of contact with eyes, rinse immediately with water or normal saline, occasionally lifting upper and lower lids until no evidence remains. Obtain medical attention.
- If swallowed, seek medical advice immediately.
- In case of accident or you feel unwell, seek medical advice immediately (show the label where possible).

Warning

H302 Harmful if swallowed.

P301+P312 IF SWALLOWED: Call a POISON CENTER/doctor if you feel unwell. Dispose of contents/container in accordance with local and national regulations.

**16CFR1500 (U.S.A.):**WARNING
HARMFUL OR FATAL IF SWALLOWED.**Examination procedure**

- Put a Fluorocell PLT cartridge in the prescribed position and then connect the Fluorocell PLT line.
- Do not remove the IC tag until disposal. All the product information is managed by the IC tag on the label.
- After setting, reset of the package is not recommended. Removing the reagent package from the analyzer may cause deterioration of the reagent by contamination and opening in a sealing film.
- Refer to the analyzer's Instruction for Use for further information.

Storage and shelf life of unopened product**Storage and shelf life after first opening**

1. Fluorocell PLT has an unopened stability of 12 months from date of manufacture when stored at 2-35°C in a dark place. See outer package or label on the cartridge for expiration date. Please do not use reagent once frozen.
2. Once opened and installed in the instrument, Fluorocell PLT is stable for 90 days.

Performance characteristics**Limitations of the examination procedure**

Performance should be within instrument specifications. Fluorocell PLT has to be used within the ambient temperature range of 15-30°C. Platelet count may be inaccurate due to the influences of abnormal samples. Consult the Instruction for Use for indications of these conditions. Confirm the platelet count by reference methods if these conditions are indicated. The use of this reagent, respectively control or calibrator, is validated on specific analyzers to optimize product performance and meet product specifications. Please refer to Instructions For Use of your analyzer whether the use of this reagent, respectively control or calibrator, is authorized by Sysmex. Sysmex cannot take the responsibility for patient results received from the use of Sysmex reagents, controls or calibrators on unauthorised analyzers. It is the responsibility of the user to validate modifications to these instructions or use of the reagent, control or calibrator on analyzers other than those specified by Sysmex.

Additional required equipment

Fluorocell PLT is intended for only use in conjunction with Sysmex lysing reagent CELLPACK DFL. Consult the Instruction for Use for specific reagent requirements.

Primary sample collection, handling and storage

Fluorocell PLT is intended for use with blood specimens collected either by venipuncture or micro-sampling by skin puncture. Specimens should be collected in EDTA anticoagulant (EDTA-K2, EDTA-K3 or EDTA-Na2). Note, that the anticoagulant EDTA-Na2 may not dissolve easily in blood, and thus causing fibrin formation or platelet aggregation in some samples. Thorough mixing is required until all dry anticoagulant is dissolved. See in the analyzer's Instruction for Use for further information regarding sample requirements.

Disposal procedures

- Tightly seal the spout of a cartridge before disposing to prevent residual reagent solution leaks. You may use tape to secure the spout.
- Disposal procedures should meet requirements of applicable local regulations.

Manufacturer

Sysmex Corporation
1-5-1 Wakino-hama-Kaigandori, Chuo-ku, Kobe 651-0073, Japan

Authorized representative / Distributors

Americas: Sysmex America, Inc.
577 Aptakisic Road, Lincolnshire, IL 60069, U.S.A.
Asia-Pacific: Sysmex Asia Pacific Pte Ltd.
9 Tampines Grande #06-18, Singapore 528735

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