Fluorocell™ WNR**Identification of the IVD reagent****Fluorocell™ WNR****Intended use**

Fluorocell WNR is to be used to label the nucleated cells in diluted and lysed blood samples for determination of white blood cell count, nucleated red blood cell count and basophil count in blood with Sysmex automated hematology analyzers.

Principles of the examination method

A sample volume of a whole blood specimen is introduced into the analyzer where a portion of it is automatically diluted into a 1:60 dilution and lysed by addition of the special Sysmex lysing reagent Lysercell WNR. Fluorocell WNR is then added and the entire dilution is maintained at a constant temperature for a defined time period in order to label the nucleated cells in the sample. The labeled sample is then introduced into the sheath flow detector where forward scattered light and side fluorescence are measured allowing the white blood cell count (WBC#), the nucleated red blood count (NRBC#), the basophil count (BASO#), the nucleated red blood percent (NRBC%) and the basophil percent (BASO%) to be computed. Consult your specific Sysmex analyzer's Instruction for Use for additional information with respect to procedures and principles of NRBC analysis.

Components

| | |
|----------------|--------|
| Polymethine | 0.005% |
| Ethyleneglycol | 99.9% |

Warnings and precautions (For in vitro diagnostic use only)

1. Wear gloves and a lab coat for protection. Avoid contact with skin and eyes.
2. In case of skin contact, rinse immediately with plenty of soap and water.
3. In case of contact with eyes, rinse immediately with water or normal saline, occasionally lifting upper and lower lids until no evidence remains. Obtain medical attention.
4. If swallowed, seek medical advice immediately.
5. In case of accident or you feel unwell, seek medical advice immediately (show the label where possible).

Warning
H302 Harmful if swallowed.
P301+P312 IF SWALLOWED: Call a POISON CENTER/doctor if you feel unwell.
P501 Dispose of contents/container in accordance with local and national regulations.

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