Fluorocell™ WPC**Identification of the IVD reagent**

Fluorocell™ WPC

Intended use

Fluorocell WPC is to be used to label the leukocytes in diluted and lysed blood samples for detection of various immature cells in blood with Sysmex automated hematology analyzers.

Principles of the examination method

A sample volume of a whole blood specimen is introduced into the analyzer where a portion of it is automatically diluted into a 1:60 dilution and lysed by addition of the special Sysmex lysing reagent Lyscell WPC. Fluorocell WPC is then added and the entire dilution is maintained at a constant temperature for a defined time period in order to label the nucleated cells in the sample. The labeled sample is then introduced into the sheath flow detector where forward scattered light, side scattered light and side fluorescence are detected to determine the presence of various immature cells (e.g. Blast and HPC). Consult your specific Sysmex analyzer's instruction for use for additional information with respect to procedures and principles of Lyscell WPC analysis.

Components

Polyethylene	0.004%
Ethanol	15.1%
Ethyleneglycol	84.8%

Warnings and precautions (For in vitro diagnostic use only)

- Wear gloves and a lab coat for protection. Avoid contact with skin and eyes.
- In case of skin contact, rinse immediately with plenty of soap and water.
- In case of contact with eyes, rinse immediately with water or normal saline, occasionally lifting upper and lower lids until no evidence remains. Obtain medical attention.
- If swallowed, seek medical advice immediately.
- Do not breath vapor. In case of accident or you feel unwell, seek medical advice immediately (show the label where possible).

Warning
H226 Flammable liquid and vapour.
H302 Harmful if swallowed.

H319 Causes serious eye irritation.
P210 Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.

P280 Wear protective gloves/protective clothing/eye protection/face protection.

P337 + P313 If eye irritation persists: Get medical advice/attention.

P501 Disposal of contents/container in accordance with local and national regulations.

16CFR1500 (U.S.A.):
WARNING
HARMFUL OR FATAL IF SWALLOWED.

Examination procedure

- Put a Fluorocell WPC cartridge in the prescribed position and then connect the Fluorocell WPC line.
- Do not remove the IC tag until disposal. All the product information is managed by the IC tag on the label.
- After setting, reset of the package is not recommended. Removing the reagent cartridge from the analyzer may cause deterioration of the reagent by contamination and opening in a sealing film.
- Refer to the analyzer's Instruction for Use for further information.

Storage and shelf life of unopened product

- Storage and shelf life after first opening**
- Fluorocell WPC has an unopened stability of 12 months from date of manufacture when stored at 2-35°C in a dark place. See outer package or label on the cartridge for expiration date. Please do not use reagent once frozen.
 - Once opened and installed in the instrument, Fluorocell WPC is stable for 90 days.

Performance characteristics

Limitations of the examination procedure

Performance should be within instrument specifications. Fluorocell WPC has to be used within the ambient temperature range of 15-30°C. Detection of immature cell may be inaccurate due to the influences of abnormal samples. Consult the Instruction for Use for indications of these conditions. Confirm the WBC differential by reference methods if these conditions are indicated. The use of this reagent, respectively control or calibrator, is validated on specific analyzers to optimize product performance and meet product specifications. Please refer to Instructions For Use of your analyzer whether the use of this reagent, respectively control or calibrator, is authorised by Sysmex. Sysmex cannot take the responsibility for patient results received from the use of Sysmex reagents, controls or calibrators on unauthorised analyzers. It is the responsibility of the user to validate modifications to these instructions or use of the reagent, control or calibrator on analyzers other than those specified by Sysmex.

Additional required equipment

Fluorocell WPC is intended for use in conjunction with Sysmex lysing reagent Lyscell WPC. Consult the analyzer's Instruction for Use for specific reagent requirements.

Primary sample collection, handling and storage

Fluorocell WPC is intended for use with blood specimens collected either by venipuncture or micro-prick by skin puncture. Specimens should be collected in EDTA anticoagulant (EDTA-K2, EDTA-K3 or EDTA-Na2). Note, that the anticoagulant EDTA-Na2 may not dissolve easily in blood, and thus causing fibrin formation or platelet aggregation in some samples. Thorough mixing is required until all dry anticoagulant is dissolved. See in the analyzer's Instruction for Use for further information regarding sample requirements.

Disposal procedures

- Tightly seal the spout of a cartridge before disposing to prevent residual reagent solution leaks. You may use tape to secure the spout.
- Disposal procedures should meet requirements of applicable local regulations.

Manufacturer

Sysmex Corporation
1-51 Wakino-hama-Kaigandori, Chuo-ku, Kobe 651-0073, Japan

Authorized representative / Distributors

Americas: Sysmex America, Inc.
577 Aptakisic Road, Lincolnshire, IL 60069, U.S.A.
Asia-Pacific: Sysmex Asia Pacific Pte Ltd.
9 Tampines Grande #06-18, Singapore 528735

Product information

Fluorocell WPC (WPC-800A)
12mL x 2 cartridges

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