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Intended use

For In Vitro Diagnostic Use

For calibration of HISCL Anti-HBe Assay Kit.

Components

1. HISCL HBeAb NC 1 mL × 1
2. HISCL HBeAb PC 1 mL × 1

Warnings and precautions

1. HISCL HBeAb PC contains anti-HBe monoclonal antibody (mouse), it does not contain human-derived materials.
2. Each Calibrator contains sodium azide. Since sodium azide reacts with lead tubing and copper tubing to generate metal azides which can explode, use a large quantity of water when disposing of it. In case of contact with the eyes, mouth, or hands, carry out emergency treatment such as washing with a large quantity of water. If necessary, consult a physician.

Examination procedure

Dispense about 200 µL (5 drops) of each Calibrator.

Carry out procedures according to the package insert for the HISCL Anti-HBe Assay Kit.

Storage and shelf life after first opening

Store at 2-8℃.

The shelf life is 90 days after opening.

Do not freeze.

Manufacturer

Japan Lyophilization Laboratory
3-1-5 Matsuyama, Kiyose-shi,
Tokyo 204-0022, Japan

Authorized representatives

Asia-Pacific:

Sysmex Asia Pacific Pte Ltd.

9 Tampines Grande #06-18, Singapore 528735

Traceability of values assigned to calibrators

HISCL HBeAb PC has been adjusted by HBe Reference Material 82 (IgG-anti HBe) made by Paul-Ehrlich-institute.

Date of issue or revision

9/2018

[REF] Catalogue number

[IVD] In vitro diagnostic medical device

Manufacturer

Consult instructions for use

Temperature limitation

Use by

[LOT] Batch code

Printed in Japan