

HISCL™ HBsAb キャリブレータ

【使用法】

1. 本品はHISCL HBsAb試薬のキャリブレータとして使用してください。
2. HISCL HBsAb試薬の添付文書に記載されている内容に従って操作してください。
3. 本品には、下記の構成試薬が含まれています。いずれも調製済みですので、そのまま使用してください。

HISCL HBsAb C0 (0 mIU/mL)	1 mL × 1本
HISCL HBsAb C1 (10 mIU/mL)	1 mL × 1本
HISCL HBsAb C2 (100 mIU/mL)	1 mL × 1本
HISCL HBsAb C3 (1000 mIU/mL)	1 mL × 1本
4. 本品は、サンプルカップに約200μL (5滴) 分注して使用してください。
5. 開封後の有効期間は90日です。

【較正用の基準物質(標準物質)】

WHO International Reference W1042

【使用上又は取扱い上の注意】

1. 本品はウサギポリクローナル抗体を原料としており、ヒト由来成分は使用していません。
2. 各構成試薬はアジ化ナトリウムを含有していますが、法的には毒物として取り扱われません。アジ化ナトリウムは、鉛、銅などと反応して爆発性の化合物を生成する危険性がありますので、廃棄の際には、大量の水と共に流してください。また、誤って目や口に入ったり皮膚に付着した場合は、水で十分に洗い流す等の応急処置を行い、必要があれば医師の手当て等を受けてください。

【貯蔵方法・有効期間】

貯蔵方法：2～8℃に保存

有効期間：12カ月（使用期限は、外箱に表示しています。）

** HISCL はシスメックス株式会社の商標です。

【製造販売元】

株式会社 日本凍結乾燥研究所

東京都清瀬市松山3丁目1番5号 〒204-0022

** TEL 042-492-5329

【販売元】

シスメックス株式会社

神戸市中央区脇浜海岸通1-5-1 〒651-0073

TEL 078-265-0500

[REF] カタログ番号

[IVD] 体外診断用の専用製品

製造販売元

添付文書参照

保存温度

使用期限

[LOT] ロット番号

**

HISCL™ Anti-HBs Calibrator

Identification of the IVD reagent

HISCL™ Anti-HBs Calibrator

Intended use

For In Vitro Diagnostic Use

For calibration of HISCL Anti-HBs Assay Kit.

Components

1. HISCL HBsAb C0 (0 mIU/mL) 1 mL × 1
2. HISCL HBsAb C1 (10 mIU/mL) 1 mL × 1
3. HISCL HBsAb C2 (100 mIU/mL) 1 mL × 1
4. HISCL HBsAb C3 (1000 mIU/mL) 1 mL × 1

Warnings and precautions

1. HISCL HBsAb C1-C3 contain Rabbit anti-HBs antigen polyclonal antibody, they do not contain human-derived materials.
2. Each Calibrator contains sodium azide. Since sodium azide reacts with lead tubing and copper tubing to generate metal azides which can explode, use a large quantity of water when disposing of it. In case of contact with the eyes, mouth, or hands, carry out emergency treatment such as washing with a large quantity of water. If necessary, consult a physician.

Examination procedure

Dispense about 200μL (5 drops) of each Calibrator.

Carry out procedures according to the package insert for the HISCL Anti-HBs Assay Kit.

Storage and shelf life after first opening

Store at 2-8℃.

The shelf life is 90 days after opening.

Do not freeze.

Manufacturer

Japan Lyophilization Laboratory
3-1-5 Matsuyama, Kiyose-shi,
Tokyo 204-0022, Japan

Authorized representatives

Asia-Pacific:

Sysmex Asia Pacific Pte Ltd.
9 Tampines Grande #06-18, Singapore 528735

Traceability of values assigned to calibrators

HISCL HBsAb C1-C3 have been adjusted by WHO International Reference W1042.

Date of issue or revision

9/2018

[REF] Catalogue number

[IVD] In vitro diagnostic medical device

Manufacturer

Consult instructions for use

Temperature limitation

Use by

[LOT] Batch code

Printed in Japan