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Intended use

For In Vitro Diagnostic Use

For calibration of HISCL Anti-TP Assay Kit.

Components

1. HISCL TP NC 1 mL × 1
2. HISCL TP PC 1 mL × 1

Warnings and precautions

1. HISCL TP PC contains human serum. Handle with care as potentially infectious, though the material has been checked as negative for HBs antigen, HCV antibody, HIV-1 antibody, and HIV-2 antibody.
2. Each Calibrator contains sodium azide. Since sodium azide reacts with lead tubing and copper tubing to generate metal azides which can explode, use a large quantity of water when disposing of it. In case of contact with the eyes, mouth, or hands, carry out emergency treatment such as washing with a large quantity of water. If necessary, consult a physician.

Examination procedure

Dispense about 200μL (5 drops) of each Calibrator.

Carry out procedures according to the package insert for the HISCL Anti-TP Assay Kit.

Storage and shelf life after first opening

Store at 2-8℃.

The shelf life is 90 days after opening.

Do not freeze.

Manufacturer

Japan Lyophilization Laboratory
3-1-5 Matsuyama, Kiyose-shi,
Tokyo 204-0022, Japan

Authorized representatives

Asia-Pacific:
Sysmex Asia Pacific Pte Ltd.
9 Tampines Grande #06-18, Singapore 528735

Traceability of values assigned to calibrators

HISCL TP PC has been adjusted by in house Anti-TP antibody Reference Material.

Date of issue or revision

9/2018

Catalogue number

In vitro diagnostic medical device

Manufacturer

Consult instructions for use

Temperature limitation

Use by

Batch code

Biological Risks

Printed in Japan