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WHO 1st International Reference Preparation, CEA 73/601

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Intended use

For In Vitro Diagnostic Use

For calibration of HISCL CEA Assay Kit.

Components

1. HISCL CEA C0 (CEA 0 ng/mL) 1 mL × 1
2. HISCL CEA C1 (CEA 5 ng/mL) 1 mL × 1
3. HISCL CEA C2 (CEA 30 ng/mL) 1 mL × 1
4. HISCL CEA C3 (CEA 100 ng/mL) 1 mL × 1
5. HISCL CEA C4 (CEA 500 ng/mL) 1 mL × 1
6. HISCL CEA C5 (CEA 1000 ng/mL) 1 mL × 1

Warnings and precautions

1. HISCL CEA C1 - C5 contains human CEA. Handle with care as potentially infectious, though the material has been checked as negative for HBs antigen, HIV-1 antibody, and HIV-2 antibody.
2. Each Calibrator contains sodium azide. Since sodium azide reacts with lead tubing and copper tubing to generate metal azides which can explode, use a large quantity of water when disposing of it. In case of contact with the eyes, mouth, or hands, carry out emergency treatment such as washing with a large quantity of water. If necessary, consult a physician.

Examination procedure

Dispense about 200 µL (5 drops) of each Calibrator.

Carry out procedures according to the package insert for the HISCL CEA Assay Kit.

Storage and shelf life after first opening

Store at 2 - 8°C. The shelf life is 90 days after opening.

Do not freeze.

Manufacturer

KAINOS LABORATORIES, INC.
38-18, Hongo 2-chome, Bunkyo-ku,
Tokyo 113-0033, Japan

Authorized representative / Distributors

Asia-Pacific: **Sysmex Asia Pacific Pte Ltd.**
9 Tampines Grande #06-18, Singapore 528735

Traceability of values assigned to calibrators and trueness-control materials

HISCL CEA C1 - C5 has been adjusted by in-house standard materials based on WHO 1st International Reference Preparation, CEA 73/601.

Date of issue or revision

05/2016

Printed in Japan

REF	Catalogue number
IVD	In vitro diagnostic medical device
	Manufacturer
	Consult instructions for use
	Temperature limitation
	Use by
LOT	Batch code
	Biological Risks