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HISCL™ Diluent

INTENDED PURPOSE

For dilution of samples measured with each assay kit.

WARNINGS AND PRECAUTIONS

1. This reagent can be used only when specified in the package insert for each assay kit.
2. This reagent contains sodium azide.
Since sodium azide reacts with lead tubing and copper tubing to generate metal azide which can explode, use a large quantity of water when disposing of it. In case of contact with the eyes, mouth, or hands, carry out emergency treatment such as washing with a large quantity of water. If necessary, consult a physician or follow in-hospital infection manual.

PROCEDURE

1. First of all push down the outer case of the reagent container firmly to tear the aluminum seals on the inner bottle.
2. Set the container at the indicated position in the analyzer.
3. Carry out procedures according to the package insert for each assay kit.

STORAGE AND SHELF LIFE AFTER OPENING

Store at 2-8°C. The shelf life is 90 days after opening.

MANUFACTURER

Sysmex Corporation
1-5-1 Wakino-hama-Kaigandori,
Chuo-ku, Kobe 651-0073, Japan


AUTHORISED REPRESENTATIVES

Asia-Pacific: Sysmex Asia Pacific Pte Ltd.
9 Tampines Grande #06-18, Singapore 528735

DATE OF ISSUE OR REVISION

08/2020

Printed in Japan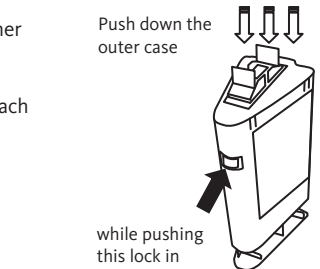

[REF]	Catalogue number
[IVD]	In vitro diagnostic medical device
	Manufacturer
	Consult instructions for use
	Temperature limitation
	Use by
[LOT]	Batch code