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Intended use

For In Vitro Diagnostic Use

For calibration of HISCL PSA Assay Kit.

Components

1. HISCL PSA C0 (PSA 0 ng/mL) 1 mL × 1
2. HISCL PSA C1 (PSA 2 ng/mL) 1 mL × 1
3. HISCL PSA C2 (PSA 20 ng/mL) 1 mL × 1
4. HISCL PSA C3 (PSA 50 ng/mL) 1 mL × 1
5. HISCL PSA C4 (PSA 100 ng/mL) 1 mL × 1
6. HISCL PSA C5 (PSA 200 ng/mL) 1 mL × 1

Warnings and precautions

1. HISCL PSA C1 - C5 contains human PSA. Handle with care as potentially infectious, though the material has been checked as negative for HBs antigen, HIV-1 antibody, and HIV-2 antibody.
2. Each Calibrator contains sodium azide. Since sodium azide reacts with lead tubing and copper tubing to generate metal azides which can explode, use a large quantity of water when disposing of it. In case of contact with the eyes, mouth, or hands, carry out emergency treatment such as washing with a large quantity of water. If necessary, consult a physician.

Examination procedure

Dispense about 200 µL (5 drops) of each Calibrator.

Carry out procedures according to the package insert for the HISCL PSA Assay Kit.

Storage and shelf life after first opening

Store at 2 - 8°C. The shelf life is 90 days after opening.

Do not freeze.

Manufacturer

KAINOS LABORATORIES, INC.
38-18, Hongo 2-chome, Bunkyo-ku,
Tokyo 113-0033, Japan

Authorized representative / Distributors

Asia-Pacific: **Sysmex Asia Pacific Pte Ltd.**
9 Tampines Grande #06-18, Singapore 528735

Traceability of values assigned to calibrators and trueness-control materials

HISCL PSA C1 - C5 has been adjusted by in-house standard materials based on WHO 1st International Standard, PSA (90:10) 96/670.

Date of issue or revision

05/2016

Printed in Japan

REF	Catalogue number
IVD	In vitro diagnostic medical device
	Manufacturer
	Consult instructions for use
	Temperature limitation
	Use by
LOT	Batch code
	Biological Risks