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INTENDED PURPOSE

For In Vitro Diagnostic Use

See package insert for each assay kit.

METHODOLOGY AND PRINCIPLE OF THE METHOD

See package insert for each assay kit.

COMPOSITION

1. HISCL R4 Reagent
2. HISCL R5 Reagent

CDP-*Star*™:Disodium 2-chloro-5-(4-methoxyspiro{1,2-dioxetane-3,2'-(5'-chloro)-tricyclo[3.3.1.1^{3,7}]decan}-4-yl)-1-phenyl phosphate 0.48mM

WARNINGS AND PRECAUTIONS

1. Use the kit according to the method stipulated in the package insert. The reliability of results cannot be guaranteed if the kit is used with a method or for a purpose other than those stipulated.
2. Avoid contact of these reagents with the skin or eyes. (R4 reagent contains sodium azide, and R5 reagent is an alkaline solution with pH9.6.)
3. Do not use the reagents once they have frozen, since they may exhibit deterioration.
4. In case of contact with the eyes, mouth, or hands, perform emergency treatment such as washing with a large quantity of water. If necessary, consult a physician.
5. Do not use the reagent bottles, etc. for other purposes.
6. Prevent contamination of alkaline phosphatase in saliva or on skin. To prevent absorption of excess CO₂, do not remove R5 reagent from the instrument until its bottle is emptied and requires replacement.

PROCEDURE

Install reagent bottles according to the instruction manual for the analyzer.

Perform procedures according to the package insert for each assay kit.

STORAGE AND SHELF LIFE AFTER OPENING

Store at 2-8°C. The shelf life is 30 days after opening.

INTERNAL QUALITY CONTROL

See package insert for each assay kit.

CALCULATION OF ANALYTICAL RESULTS AND REFERENCE INTERVALS

See package insert for each assay kit.

PERFORMANCE CHARACTERISTICS AND LIMITATIONS OF THE METHOD

See package insert for each assay kit.

REAGENT PREPARATION

All reagents are ready-to-use.

SPECIMENS

See package insert for each assay kit.

PROCEDURE FOR DISPOSAL

1. R4 reagent contains sodium azide. Since sodium azide reacts with lead tubing and copper tubing to generate metal azides which can explode, use a large quantity of water when disposing it.
2. Incinerate used reagent bottles, or dispose of them as medical waste or industrial waste according to the rules stipulated for waste materials.

LITERATURE REFERENCES

See package insert for each assay kit.

MANUFACTURER

 Sysmex Corporation
1-5-1 Wakinohama-Kaigandori, Chuo-ku,
Kobe 651-0073, Japan

AUTHORIZED REPRESENTATIVE/DISTRIBUTOR

Asia-Pacific: Sysmex Asia Pacific Pte Ltd.
9 Tampines Grande #06-18, Singapore 528735

PRODUCT INFORMATION

HISCL Substrate Reagent Set (NBS-900A)

HISCL R4 Reagent 40 mL × 1 bottle per box

HISCL R5 Reagent 70 mL × 1 bottle per box

TRACEABILITY OF CALIBRATORS AND CONTROL MATERIALS

See package insert for each assay kit.

DATE OF ISSUE OR REVISION

02/2017

Printed in Japan

	Catalogue number		Use by
	In vitro diagnostic medical device		Batch code
	Manufacturer		
	Consult instructions for use		
	Temperature limitation		

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