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HISCL™ Troponin T hs Calibrator

Intended use

For in vitro diagnostic use only
For calibration of HISCL Troponin T hs Assay Kit

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Warnings and precautions

- 1) HISCL Troponin T hs C1-C5 contain human serum. Handle with care as potentially infectious, though the calibrator material has been checked as negative for HBs antigen, HCV antibody, HIV antibody.
- 2) In case of contact with eyes, mouth or hands, perform emergency treatment such as washing with a large quantity of water. If necessary, consult a physician or follow in-hospital infection manual.

Examination procedure

Add 1 mL of distilled water to Troponin T hs C0-C5, put the cap on them, and leave for 15 minutes. Mix by inversion with care not to foam the solution to dissolve the content into the solution completely. Dispense about 200 μ L of each calibrator. Perform procedures according to the package insert for the HISCL Troponin T hs Assay Kit.

Storage and shelf life after first opening

Store at 2-8°C before reconstitution. Do not freeze.
Store below -20°C after reconstitution.
The shelf life is 90 days after reconstitution.
Freezing and thawing can be repeated 3 times.

Manufacturer

Sysmex Corporation

1-5-1 Wakinohama-Kaigandori, Chuo-ku, Kobe 651-0073, Japan

Authorized representatives

Asia-Pacific: Sysmex Asia Pacific Pte Ltd.
9 Tampines Grande #06-18, Singapore 528735

Traceability of values assigned to calibrators

HISCL Troponin T hs C1-C5 have been adjusted by in-house standard materials.

Reference values

The reference values are indicated on the reverse side of this document.
Set these values according to the instruction manual for the instrument used.

Date of issue or revision

05/2020

Symbol

Refer to the HISCL Instructions for Use for the details of the symbols written on the outer box.

Sysmex Corporation

Printed in Japan

2
AN418118D