Cat. No. 984-1721-6 , 054-3331-0

## STROMATOLYSER® -4DS

**PRODUCT NAME**  
**STROMATOLYSER® -4DS****INTENDED PURPOSE (FOR IN VITRO DIAGNOSTIC USE ONLY)**

**STROMATOLYSER-4DS** is to be used to stain the leukocytes in diluted and lysed blood samples for determination of 4-part differential count, or 5-part differential count\* and WBC count\* with Sysmex automated hematology analyzers (\*on XS series only).

**METHODOLOGY AND PRINCIPLE OF THE METHOD**

A sample volume of a whole blood specimen is introduced into the analyzer where a portion of it is automatically diluted into a 1:50 dilution and lysed by addition of the special Sysmex lysing reagent **STROMATOLYSER-4DL**. **STROMATOLYSER-4DS** is then added and the entire dilution is maintained at a constant temperature for a defined time period in order to stain the nucleated cells in the sample. The stained sample is then introduced into the sheath flow detector where side light scatter and side fluorescent emission are measured allowing the WBC count (WBC#)\* the neutrophil count (NEUT#), the lymphocyte count (LYMPH#), the monocyte count (MONO#), the eosinophil count (EO#), the basophil count (BASO#)\* the neutrophil percent (NEUT%), the lymphocyte percent (LYMPH%), the monocyte percent (MONO%), the eosinophil percent (EO%) and the basophil percent (BASO%)\* to be computed. Consult your specific Sysmex instrument Operator's Manual for additional information with respect to procedures and principles of 4-part differential analysis, or 5-part differential analysis\* and WBC count analysis\* (\*on XS series only).

**COMPOSITION**  
**STROMATOLYSER-4DS** (FFS-800)

|                    |        |
|--------------------|--------|
| Polymethine dye    | 0,002% |
| Methanol           | 3,0%   |
| in ethylene glycol | 96,9%  |

**WARNINGS AND PRECAUTIONS**

- 1.Wear gloves and a lab coat for protection. Avoid contact with skin and eyes.
- 2.In case of skin contact, rinse immediately with plenty of soap and water.
- 3.In case of contact with eyes, rinse immediately with water or normal saline, occasionally lifting upper and lower lids until no evidence of dye remains. Obtain medical attention.
- 4.If swallowed, seek medical advice immediately.
- 5.In case of accident or you feel unwell, seek medical advice immediately (show the label where possible).

**R20/22** : Harmful by inhalation and if swallowed.

**S7** : Keep container tightly closed.

**S16** : Keep away from sources of ignition — No smoking.

**S24** : Avoid contact with skin.

**S45** : In case of accident or if you feel unwell, seek medical advice immediately (show the label where possible).

Sysmex products are labeled with Hazard Symbol and Risk and Safety Phrases in compliance with the European Community Directive.

**PROCEDURE**

- 1.Loosen and remove the cap on the **STROMATOLYSER-4DS** bag. Attach the dispenser kit to the dye bag. Tighten the cap. Connect the **STROMATOLYSER-4DS** line from the instrument to the dispenser kit.
- 2.Refer to the instrument Operator's Manual for further information.

**STORAGE AND SHELF LIFE AFTER FIRST OPENING**

**1.STROMATOLYSER-4DS** has an unopened stability of 12 months from date of manufacture when stored at 2-35°C in a dark place. See outer package or label on the bag for expiration date. **STROMATOLYSER-4DS** displaying any signs of contamination or instability, as indicated by cloudiness or colour change, should be replaced. Please do not use reagent once frozen.

2.Once opened and installed in the instrument. **STROMATOLYSER-4DS** is stable for 90 days.

**PERFORMANCE CHARACTERISTICS AND LIMITATIONS OF THE METHOD**

Performance should be within instrument specifications.

**STROMATOLYSER-4DS** has to be used within the ambient temperature range of 15-30°C. 4-Part differential count, or 5-part differential count\* and WBC count\* may be inaccurate due to the influences of abnormal samples. Consult the Operator's Manual for indications of these conditions. Confirm the WBC differential by reference methods if these conditions are indicated (\*on XS series only).

**REAGENTS REQUIRED BUT NOT PROVIDED**

**STROMATOLYSER-4DS** is intended for only use in conjunction with Sysmex lysing reagent **STROMATOLYSER-4DL**. Consult the instrument Operator's Manual for specific reagent requirements.

**SPECIMEN**

**STROMATOLYSER-4DS** is intended for use with blood specimens collected either by venepuncture or micro-sampling by skin puncture. Specimens should be collected in EDTA anticoagulant (EDTA-K<sub>2</sub>, EDTA-K<sub>3</sub> or EDTA-Na<sub>2</sub>).

Note, that the anticoagulant EDTA-Na<sub>2</sub> may not dissolve easily in blood, and thus causing fibrin formation or platelet aggregation in some samples. Thorough mixing is required until all dry anticoagulant is dissolved. See in the instrument Operator's Manual for further information regarding sample requirements.

**DISPOSAL PROCEDURES**

Disposal procedures should meet requirements of applicable local regulations.

**MANUFACTURER**

 **SYSMEX CORPORATION**  
1-5-1 Wakinohama-Kaigandori, Chuo-ku, Kobe 651-0073, Japan

**AUTHORISED REPRESENTATIVE / DISTRIBUTORS**

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